
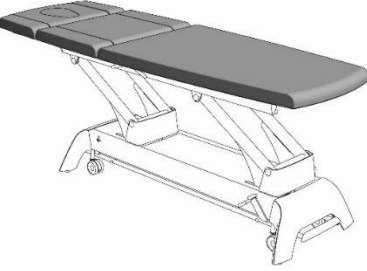
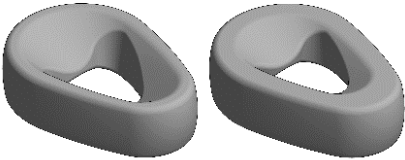


	General	Cleaning	Disinfecting
Construction 	Materials: <ul style="list-style-type: none"> • Steel or aluminium powder coated • Steel, chrome or sink plated • Hard and soft plastics • Pressed and glued beech veneer (gymna.ONE) 	<ul style="list-style-type: none"> • Clean the surfaces regularly with a clean soft and dry microfiber cloth. • For persisting stains, a slightly damp cloth may be used. • Use clean water or glass cleaner. • Remove any spilled fluids immediately. 	<p>When needed, the construction can be disinfected using the same method as for the cushions or Ergomax.</p>
Cushions 	Materials: <ul style="list-style-type: none"> • Vinyl artificial leather 'Classique' • Polyurethane artificial leather 'Premium' Details: <ul style="list-style-type: none"> • Phthalaat-free • Biocompatible • Premium is optional for gymna.PRO 	<ul style="list-style-type: none"> • Use lukewarm water and a microfiber cloth for cleaning. • Add a PH neutral soap solution in the case of persistent stains. • Remove the soap residue with a damp cloth. 	<p>Important: Clean before disinfecting.</p> <p>The cushions and Ergomax can be disinfected with a large number of disinfectants. The approved disinfectants are listed on page 2.</p>
Ergomax 	Material: <ul style="list-style-type: none"> • Polyurethane foam Details: <ul style="list-style-type: none"> • Waterproof and easy to clean at the tap. 	<ul style="list-style-type: none"> • Bloodstains must be removed with cold water immediately. • Massage oil must be removed with a PH neutral soap solution immediately. 	<ul style="list-style-type: none"> • Always follow the prescribed concentration as listed or as mentioned on the packaging. • Only use one disinfectant and rinse with clear water.
Extra guidelines	<ul style="list-style-type: none"> • NEVER use detergents, solvents (alcohol), aggressive soaps, bleaching agents or abrasives. • Do not use any substances for leather containing oil or grease for maintenance. • Never use a mixture of various disinfectants. • The cushions and Ergomax are not suitable for disinfection or sterilization by means of heat. 		

Disinfectant	Classique	Premium
Acryl-des	●	
Actichlor Plus		● 5000 ppm
Actrill Cold Sterilant		● 100%
Aldasan 2000	●	
Alpet D2	●	
Antifect FD 10	●	
Antigone wipes	●	
Antiseptica Kombi Flächen	●	
B10	●	
Bacillol 30 foam	●	● 100%
Bacillol 30 tissues		●
Bacillol AF	●	
Bacillol AF-tissues	●	
Biguacid Plus	●	
Biguacid-S	●	
Biguanid Fläche N	●	● 4%
Chlor-Clean tablets		● 5000 ppm
Cleanisept	●	
Clinell Alcohol wipes	●	
Clinell Sporicidal wipes		●
Clinell Universal wipes	●	
Clorina	●	
CosiMed	●	
Descocid-N	●	
Descogen Liquid	●	
Descosept AF	●	
Desomed Rapid Universal	●	
Dettol		● 5%
Dismozon pur		● 4%
Esemfix	●	
Ethanol		● 75%
Formaldehyde		● 30%
Fugaten	●	
Germatol		● 100%
Gevisol	●	
Haz-Tabs		● 5000 ppm
Hexaquart plus	●	
Hexaquart SF	●	
Hospisept - wipes	●	
Hydrogenperoxide		● 30%

Disinfectant	Classique	Premium
Immix Clean	●	
Incidin Extra N		● 5%
Incidin foam		● 100%
Incidin OxyFoam		● 100%
Incidin OxyFoam S		● 100%
Incidin OxyWipe S		●
Incidin Plus		● 4%
Incidur Active	●	
Incidur Extra	●	
Incidur Extra N	●	
Incidur Foam	●	
Incidur Liquid Spray	●	
Incidur Perfekt	●	
Incidur Plus	●	
Incidur rapid	●	
Incidur Spray	●	
Isopropanol		● 45%
Joya	●	
Kohrsolin extra tissues	●	
Lysoformin	●	
Lysoformin 2000	●	
Lysoformin spezial	●	
Maxitücher Alkoholfrei		●
Maxitücher Gebrauchsfertig Ruck		●
Meliseptol	●	● 100%
Meliseptol foam pure + fresh	●	
Meliseptol rapid	●	
Melsept SF	●	
Melsitt	●	
Micro Clean	●	
Micro Clean Foam	●	
Mikrobac forte	●	
Mikrobac tissues	●	
Mikrozid AF liquid	●	
Mikrozid AF wipes	●	●
Mikrozid sensitive liquid	●	
Mikrozid sensitive wipes	●	
Mikrozid universal wipes premium		●

Disinfectant	Classique	Premium
Neoform M rapid	●	
Optisal N	●	
Optisept	●	
P3-topax 66		● 5%
Perform	●	
Pursept-A Xpress	●	
Pursept-FD	●	
Sagrotan Hygiene-spray	●	● 100%
Sani-Cloth Active - wipes	●	
Sanit P20		● 100%
Septapin Aquaquick	●	
Septapin wipes	●	
Sodium hypochlorite		● 5000 ppm
Sprint 200		● 2%
Suma bac D10		● 1%
Suma Tab D4	●	
Surfa Safe	●	
Surfa Safe Premium		● 100%
Surfanios Premium		● 3%
Terralin liquid	●	
Terralin protect	●	● 4%
Topmax 421		● 5%
TPH protect	●	
Trichlorol	●	
Ultrasol active		● 5%
Umonium medical spray		● 100%
Virkon		● 1%
Volvone		● 40%

Disclaimer

GymnaUniphy NV does not accept any responsibility with regards to the disinfectant effect of the agents listed above.

© 2020 GymnaUniphy NV

All rights reserved. Nothing in this publication may be multiplied in an automated data file or published in any form or in any way, be it electronically, mechanically, by photocopy, recording or any other way, without prior written consent by GymnaUniphy N.V.